



## Installing Waste Outlet (one supplied with your machine, additional outlets can be purchased from us)

**Stage 1:** Set your bed to a comfortable working height

**Stage 2:** Place the waste trap against the wall with the top of the trap about 25-45 mm below the underneath side of your bed with the left hand outlet in the centre of the bed as shown in Fig1

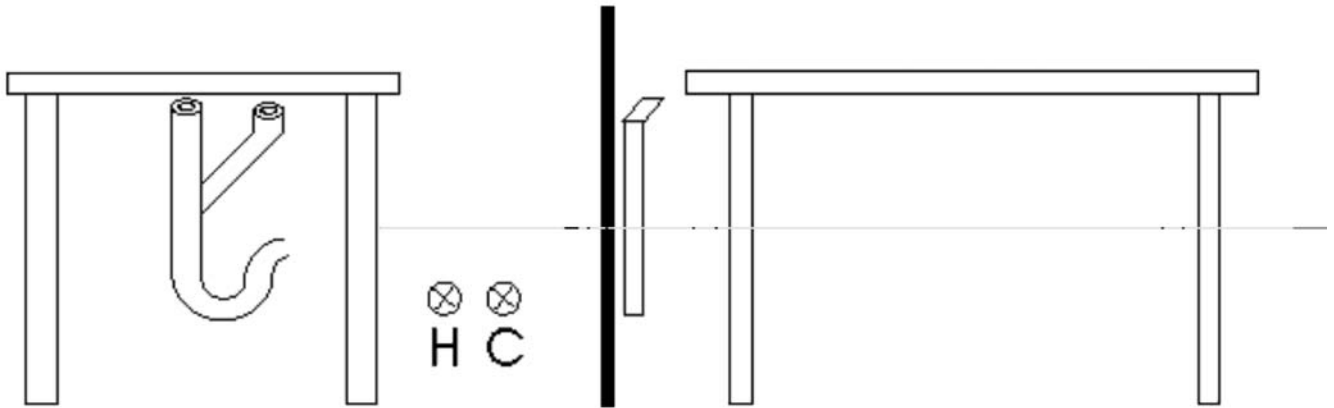


Fig 1

**Stage 3:** Fix waste trap to wall and connect to the building waste system

**Stage 4:** Install hot and cold feed to right hand side of bed

We recommend that the wall behind the fittings should be protected from water, for example by tiles or Perspex.

### CONSULTATIONS (UK ONLY)

Installation advice for your equipment is part of your training course (one section of the practical module).

***We do not consult over the phone*** because each venue has individual requirements and water supplies vary considerably.

### SITE VISITS TO ADVICE ON SUITABILITY OF YOUR VENUES (UK ONLY)

We can arrange site visits to advice on venue suitability. The fees are as follows:

£350 per working day ( 09.30 to 17.30). We will visit as many of your sites as possible within these hours. You must be present at all times.

**Additional expenses:** travel, hotel accommodation etc – please email us on [request@aquaclinic.co.uk](mailto:request@aquaclinic.co.uk) for an estimate.

All fees and expenses are subject to VAT.

For all international requests please email us.